


Ships in 3 boxes



GS 900 on Mobile Stand Assembly Instructions




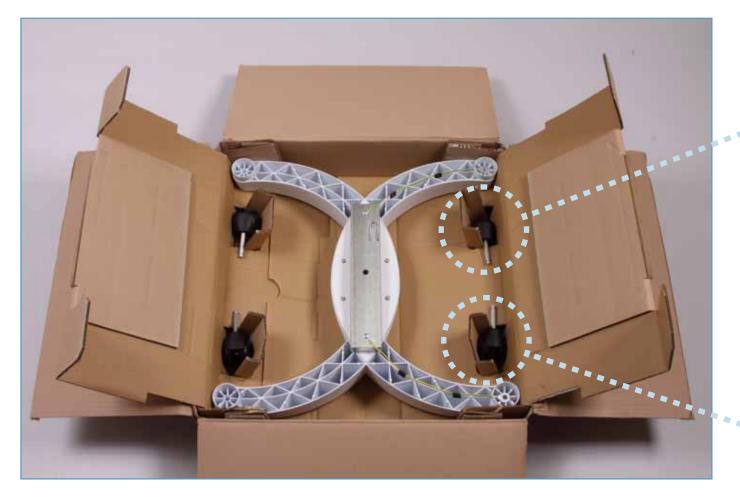


(2) wheels without brakes



(2) wheels with brakes

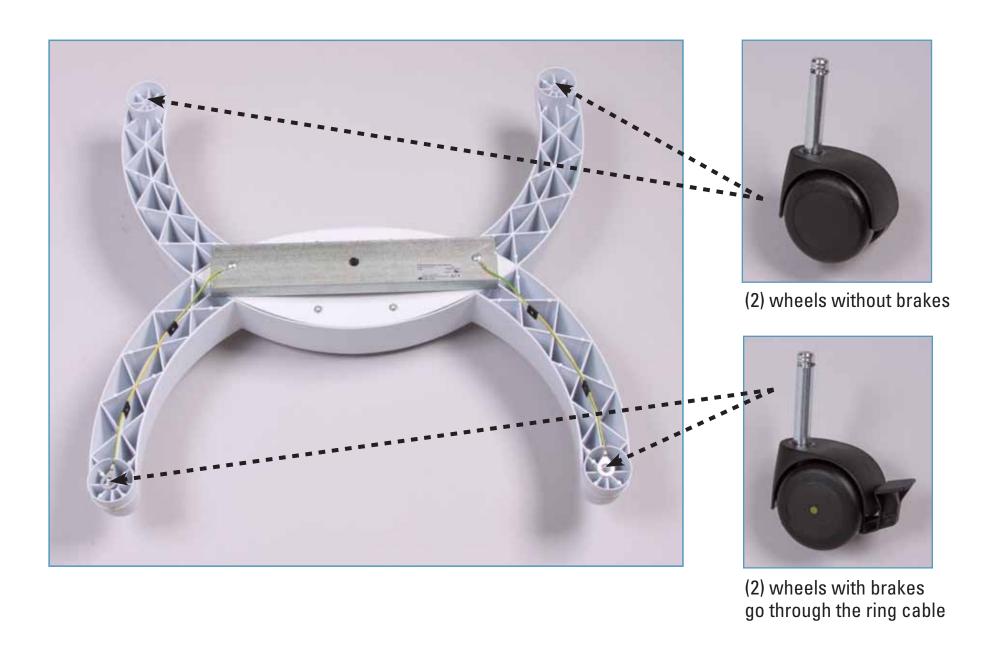

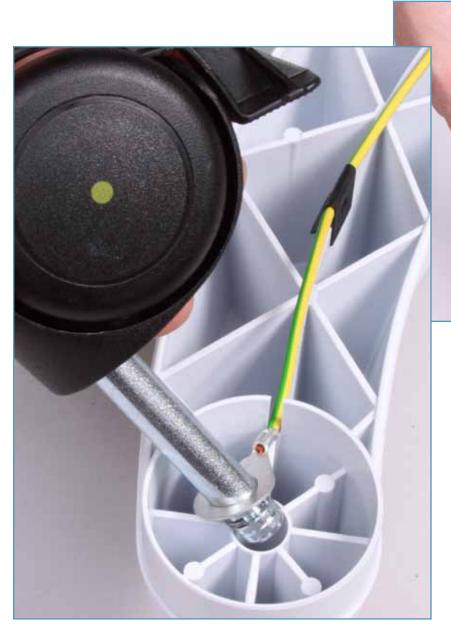

Place wheels with brakes through ring cable and insert completely






Insert two wheels without brakes completely






Parts included with Mobile Stand

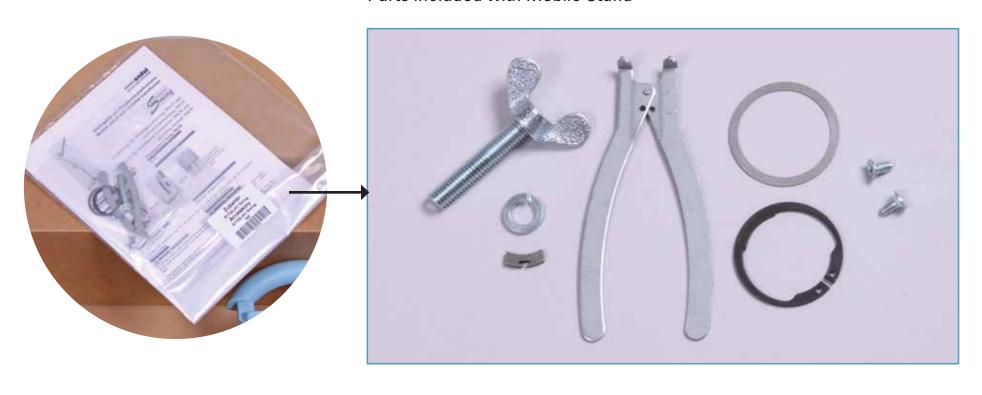




Tools not included

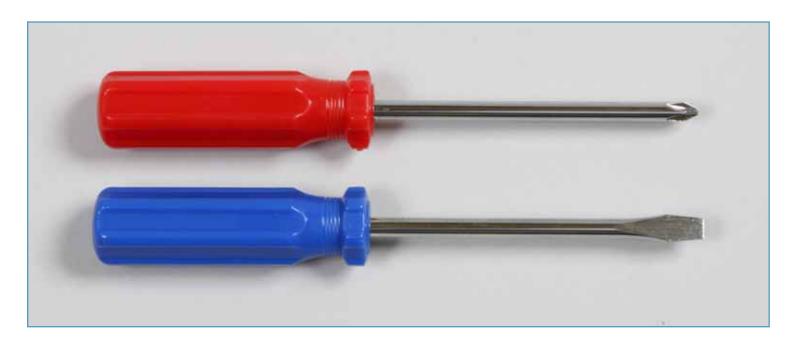



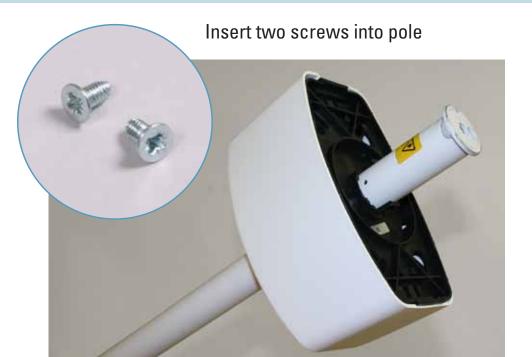















GS 900 on Mobile Stand Assembly Instructions

Insert the tripod tube so that the recess in the tube is seated in the nose of the tripod base and cannot be rotated any more























Remove and discard grey protective cover



Unscrew the Phillips/plus-slotted screw, pull the cover off, pull cord through the top opening and insert the spring arm











Place the disk over spring arm










GS 900 on Mobile Stand Assembly Instructions

Place retaining ring over spring arm and secure into the pin groove







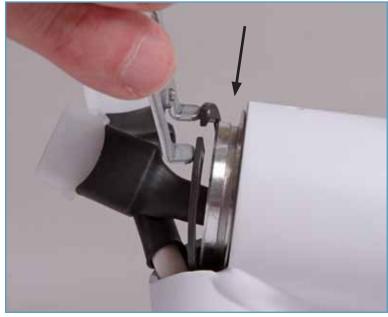





Make the electrical connection, tuck wires back inside, replace cover and tighten Phillips/plus screw























Remove and discard grey protective cover

Loosen the slotted/minus screw





Loosen the Phillips/plus-slotted screw





Luminaire



Insert luminaire into the spring arm









Place securing segment completely into slot




Tighten the Phillips/plus-slotted screw, turn plastic sleeve 180° and tighten the slotted/minus screw









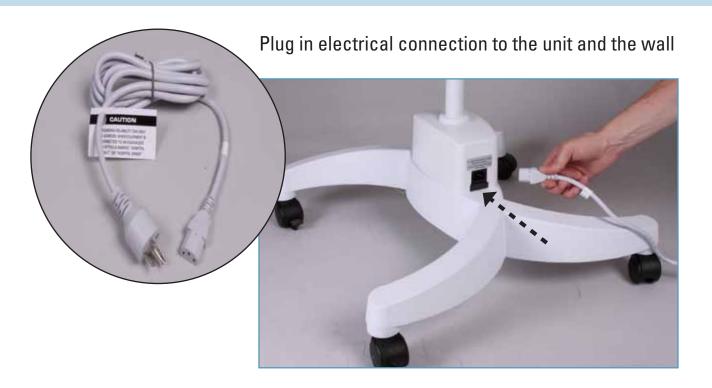





Turn on light







© 2010 MC7635