Material No. 729485 DIR No. 80017989 Rev. E. Revision date: 2019-09
 © 2019 Welch Allyn, Inc.

Rechargeable Battery Handles and Chargers
 Following this product, please read the warnings and instructions.

General Use Instructions

Maintenance

Like any other piece of electro-medical equipment, perform periodic electrical inspections by qualified personnel. Welch Allyn recommends that this equipment is inspected at least every six months, more often if used under adverse conditions.

Instructions for Recycling

Nickel-Cadmium Batteries

Welch Allyn now employs the services of an agency which can disassemble and recycle all components of nickel-cadmium batteries so that nothing gets landfilled or incinerated. Should you wish to dispose of your expended Welch Allyn rechargeable battery via recycling (in the U.S. and Canada only), please send to:

Welch Allyn Canada
RECYCLE BATTERY
 4341 State Street Road
 Skaneateles Falls, NY
 13153-0220
 For locations outside the U.S. and Canada, consult your local recycling authority for correct disposal of batteries.

Service:

If the product fails to function properly or if assistance, service, or spare parts are required, contact the nearest Welch Allyn Technical Support Center.

Welch Allyn Technical Support: <http://www.welchallyn.com/about/company/locations.htm>

Garantie

Welch Allyn nickel-cadmium batteries are guaranteed by Welch Allyn for two years from date of manufacture (when used in Welch Allyn instruments only). Defective batteries are replaced on a pro rata basis should failure occur prior to expiration date on battery.

The Welch Allyn products listed on this document are guaranteed by Welch Allyn against all manufacturing defects. Welch Allyn will repair or replace, free of charge, any parts of its own manufacture proven to be defective through cause other than misuse, neglect, damage in shipment, or normal wear.

Welch Allyn warrants that the products will perform to original specifications during the two years period of this warranty. This warranty does not apply to the use of the 7200C or 7230B battery in other products.

Conforms with: IEC/UL/CSA/EN 60601-1

Handles

Safety and Warnings

•Continuous Operation: This product complies with current required standards for electromagnetic interference and should not present problems to other equipment or be affected by other devices. As a precaution, avoid using this device in close proximity to other equipment.

• POSSIBLE EXPLOSION HAZARD. Equipment not suitable for use in the presence of a flammable anesthetic.

Do not disassemble or modify the battery pack. There are no serviceable parts inside. Do not charge battery pack in any environment where temperature may exceed 40°C (104°F) or fall below 0°C (32°F).

For longest service, remove the handle and AC Charging Module from outlet when charging is complete. Do not leave AC Charging Module connected to power outlet without a handle attached.

Do not sterilize the 710 Series Power Handle.

For information about electromagnetic compatibility (EMC), see the Welch Allyn website: www.welchallyn.com/emc-hnds.

A printed copy of the Emissions and Immunity information can be ordered from Welch Allyn for delivery within 7 calendar days.

FOR BEST PERFORMANCE:

Do not expose battery pack or storage temperatures above 50°C (122°F) or below -20°C (-4°F).

Do not discharge battery pack if the ambient temperature is above 60°C (140°F) or below -20°C (-4°F).

Nickel-Cadmium Handle

The 710 is a universal handle which accepts all Welch Allyn 3.5v instrument heads. Its adjustable rheostat control allows optimal intensity and battery life. The chrome-plated brass handles and metal head connection provide durable, long life construction and the smooth, non-abrasive knurled finish insures a no-slip grip.

The 710 series also have a built in charger for easy, overnight charging of the long-lasting rechargeable nickel-cadmium batteries. As an option, the converter ring is available in the 7100C handle, which enables insertion of off-the-shelf C-cell batteries in place of the rechargeable batteries for continued operation without the need to wait for recharging.

Specifications:

Dimensions: 5.5" Long x 1.50" Diameter (max)
 Weight: 10 oz. (including battery)
 Input: 120V 60Hz 0.17A

Before using, follow these instructions:

- fully charge battery (16 hours)
- insert replacement battery "button" side up
- fully charge (16 hours)
- run battery completely down
- fully charge (16 hours)
- ready for use


Operating Instructions:

- Connect instrument head to the handle.
- Press On/Off Button on Rheostat Section and rotate the Light Intensity Control Ring clockwise (CW). To intensify light, continue to rotate until stop is reached.

To turn off, rotate counter-clockwise (CCW) button will click when fully off. Turn fully off after each use to ensure maximum on-time from the battery.

Charging Instructions for Rechargeable Handles with Self-contained Charger

- Remove the instrument head and unscrew top cap from battery housing.
- Plug into standard (110-120v/50/60 Hz) wall outlet. Charge for 16 continuous hours when new, or after a long period of activity.
- For subsequent recharging, a single overnight charge is sufficient.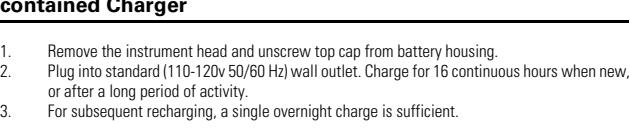

How to Replace Rechargeable Batteries

When a rechargeable battery will no longer accept a charge, replace as follows:

- Unscrew bottom cap.
- Shake out old battery.
- Insert replacement battery "button" side up.

CAUTION: Insert battery fully seated. For convertible handles, match polarity markings with those on instrument when inserting replacement battery.

Press bottom cap back into place. Ensure the plastic insulator cap is securely attached to the bottom cap. If the insulator is missing or broken, contact Welch Allyn for a new bottom cap.

For models 71000, 71010, 71020, 71050, 70700, 70710, 70720, and 70750 (non-convertible, rechargeable handles with self-contained charger), insure that plastic insulator cap is intact and securely seated on spring. If missing or damaged, install new before using.

DO NOT INTERCHANGE BOTTOM CAPS BETWEEN DIFFERENT MODELS OF RECHARGEABLE HANDLES. USE OF INCORRECT BOTTOM CAP MAY DAMAGE THE BATTERY OR CAUSE THE HANDLE TO OVERHEAT. BURNS MAY OCCUR FROM AN OVERHEATED HANDLE.

Order replacement batteries from any Welch Allyn distributor.

Welch Allyn Convertible Handles

To interchange rechargeable battery with standard ("C")

- Unscrew transformer module assembly.
- Shake out rechargeable battery.
- Insert battery converter. Product No. 710168-501 for 71000-C handle or Product No. 710168-502 for 71020-C handle (export).

Unscrew bottom cap and insert two ("C") batteries with plus (+) terminal first.

Replace bottom cap.

To interchange standard ("C") batteries with rechargeable battery, reverse above process.

Special Notes

- Slight heating of handles is normal during recharge cycle.
- Batteries cannot overheat or overcharge.
- Batteries will not leak.
- Batteries will discharge gradually over a period of about 60 days when stored at room temperature (70°F). Storage at higher temperatures accelerates the discharge rate.
- For rechargeable handles that accommodate "C" or "AA" size batteries — remove batteries from handle if instrument is not in use for an extended period of time.

Cleaning and Disinfection for Handles

Wipe the Handle with an appropriate health care low- or intermediate-level cleaner/disinfecting wipe that incorporates either a 1:10 sodium hypochlorite (bleach) solution or isopropyl alcohol as the active disinfectant ingredient. Follow wipe manufacturer's instructions for appropriate use, contact times and applicable warnings and precautions.

Do not excessively soak/saturate the Handle assembly. Do not immerse handle assembly in any solution. Do not sterilize the Power Handle.


After disinfection, inspect the Handle assembly for visible signs of deterioration at its switch, device end connector and housing. If evidence of damage or deterioration is present, discontinue use and contact Welch Allyn or your area representative.

Symbols: Environment	Symboles : Environnement	Symbole: Ophthalmoskop-Umgebungsbedingungen	Simboli: Ambiente	Symbols: Manufacturer & Date of Manufacture (YYY-MM-DD)	Symboles : Fabricant et date de fabrication (AAAA-MM-JJ)	Symbole: Manufacturer & Date of Manufacture (YYYY-MM-DD)	Simboli: Produttore e data di produzione (AAAA-MM-GG)
Temperature limitation Operating: 59°F (10°C) to 104°F (49°C) Transport/storage: -4°F (-20°C) - 120°F (55°C)	Plage de température Fonctionnement: 10 à 49 °C Transport/stockage: -20 à 55 °C	Zulässige Luftfeuchtigkeit Betrieb: 30% - 90% Transport/Lagerung: 10% - 95%	Limiti di umidità Funzionamento: 30% - 90% Transporto/conservazione: 10% - 95%	Manufacturer & Date of Manufacture (YYY-MM-DD)	Fabricant et date de fabrication (AAAA-MM-JJ)	Manufacturer & Date of Manufacture (YYYY-MM-DD)	Produttore e data di produzione (AAAA-MM-GG)
Humidity limitation Operating: 30% - 90% Transport/storage: 10% - 95%	Plage d'humidité Fonctionnement: 30 % - 90 % Transport/stockage: 10 % - 95 %	Zulässiger Luftdruck 500 hPa bis 1060 hPa	Limiti di pressione atmosferica 500 hPa - 1060 hPa	R ONLY Professional use only	À usage professionnel uniquement	Professional use only	Solo per uso professionale
Atmospheric pressure limitation 500 hPa - 1060 hPa	Plage de pression atmosphérique 500 hPa bis 1060 hPa			EC REP Authorized Representative in the European Community	Représentant autorisé dans la Communauté européenne	Authorized Representative in the European Community	Rappresentante autorizzato nella Comunità europea
Attention: Read Operating Manual for Cautions and Instructions for Use.	Attention: Lire le manuel d'utilisation pour connaître les mises en garde et consignes d'utilisation.	Vorsicht: Vorichtsmaßnahmen und Gebrauchsanweisung in der Bedienungsanleitung beachten	Attenzione: Leggere il manuale di funzionamento per conoscere le avvertenze e le istruzioni d'uso	GTIN Global Trade Item Number	Référence de commerce international	Global Trade Item Number	Numero articolo per il commercio globale
Type BF applied part	Pièce appliquée de type BF	Anwendungsteil vom Typ BF	Assistenza tecnica Welch Allyn	LOT Batch code	Codice di lotto	Reorder number	Consultare le istruzioni operative.
Class II equipment	Équipement de classe II	Gerät der Klasse II	Apparecchiatura di classe II	IPX0 Equipment not protected against the ingress of water.	Matériel non étanche.	Das Produkt ist nicht wasserdicht.	Apparecchiatura non protetta contro l'ingresso di acqua.
Reorder number	Référence de commande	Reorder number	Apparecchiatura di classe II				
Product Identifier	Identifiant du produit	Product Identifier	Identificatore del prodotto				

Battery Charger

Charger Instructions

- Plug into a 110-130 V, 50-60 Hz A.C. receptacle for No. 71040, (90-110 V for 71030, 220-250 V for 71022, 71034, 71035, and 71036).
- Connect the charging plug to charger socket on instrument or charging stand.
- Support the instrument during charging. Do not suspend the instrument by its charging cord.
- Batteries do not overcharge. A completely discharged battery is fully charged overnight.
- Use the charger with either 2.5 V or 3.5 V rechargeable instruments.
- Do not use diagnostic instruments while charging.

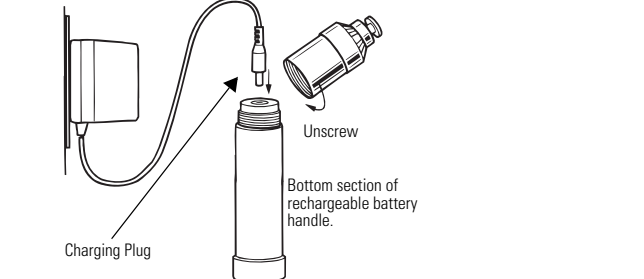
AudioScope™ 3 with Charging Stand & CompaSet

- 71123 Charging Stand for AudioScope™ 3
- 23000 AudioScope™ 3
- 92100/93410 CompaSet™ Diagnostic Set

Charging Instructions for Rechargeable Handles for Desk Charger or Pocket Set Charger

- To recharge, place handle in charging well, ensuring that contact is made at the base.
- For full capacity, charge for 16 continuous hours when brand new. For subsequent recharging, place handles in well whenever not in use or when charge is depleted.

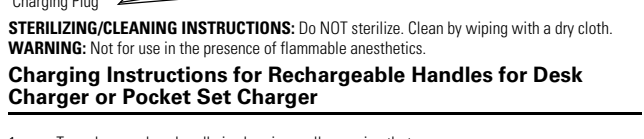STERILIZING/CLEANING INSTRUCTIONS: Do NOT sterilize. Clean by wiping with a dry cloth. **WARNING:** Not for use in the presence of flammable anesthetics.

Charging Instructions for Rechargeable Handles for Desk Charger or Pocket Set Charger

- To recharge, place handle in charging well, ensuring that contact is made at the base.
- For full capacity, charge for 16 continuous hours when brand new. For subsequent recharging, place handles in well whenever not in use or when charge is depleted.

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