

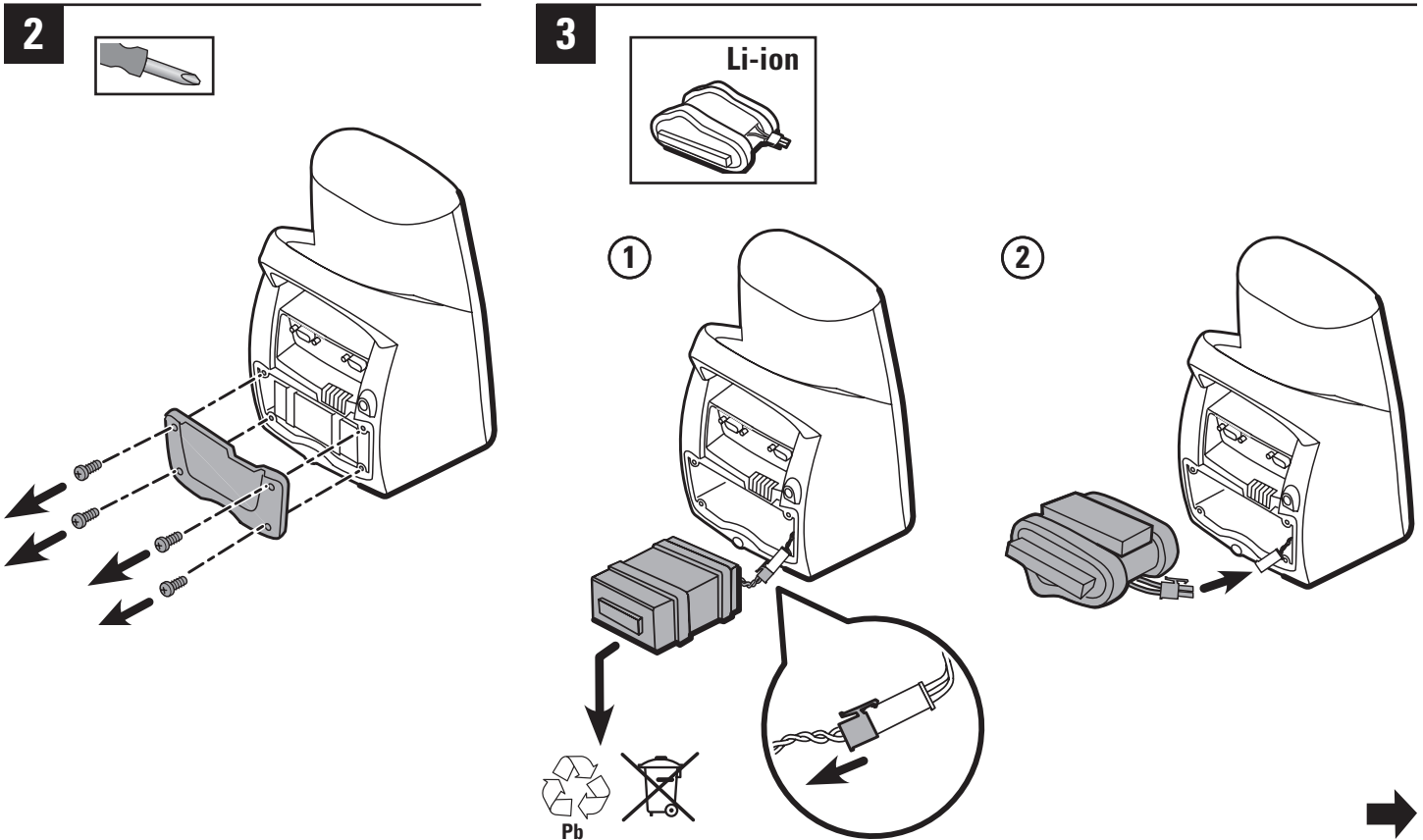
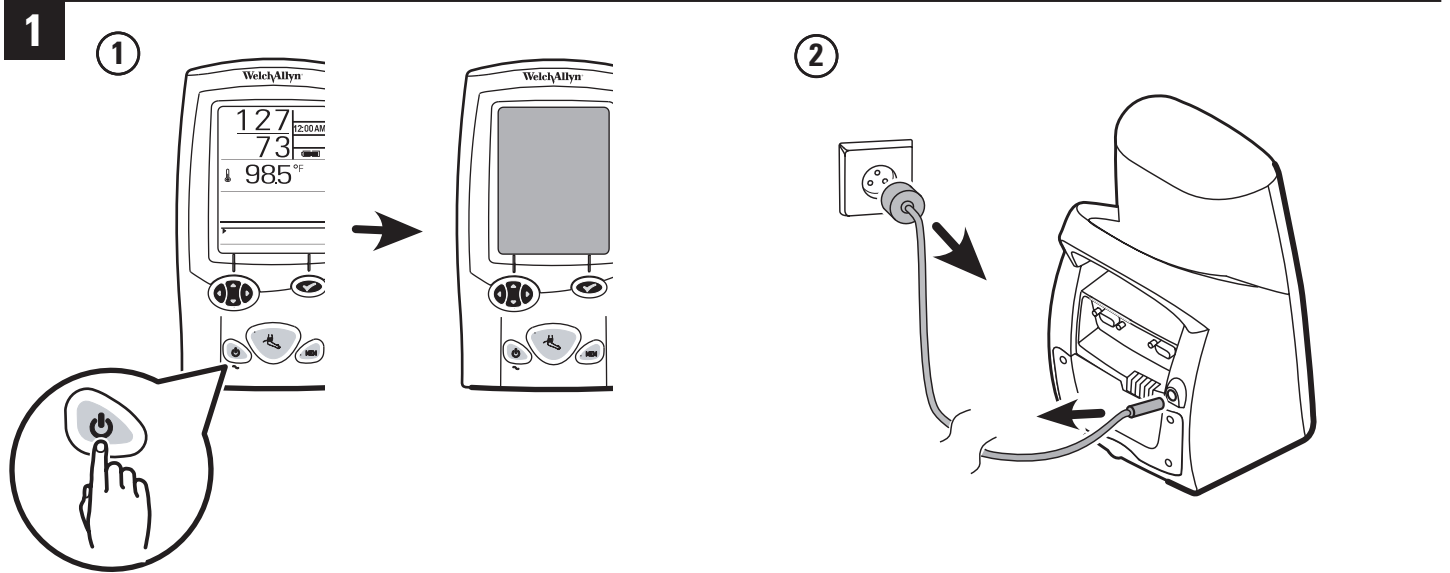
WelchAllyn®

Spot Vital Signs® LXi Battery Upgrade Install Instructions



Mat. 720745, 80018656 Ver. C
© 2013 Welch Allyn, Inc.
www.welchallyn.com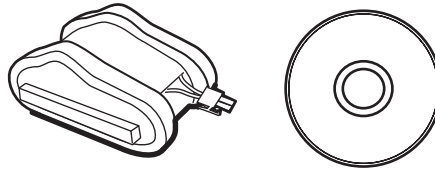


4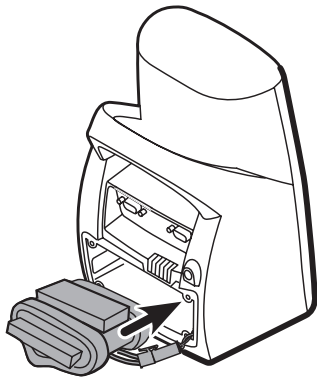


5

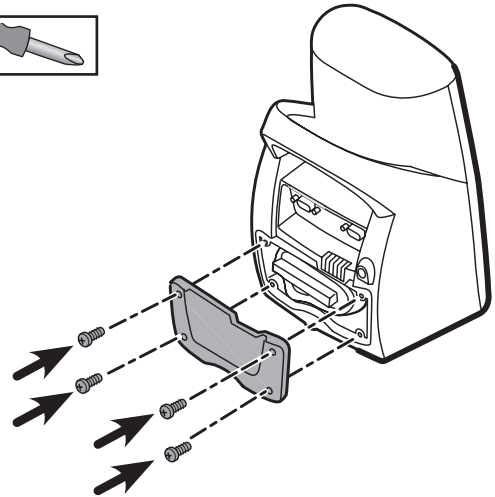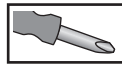6

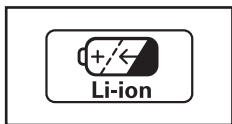
1


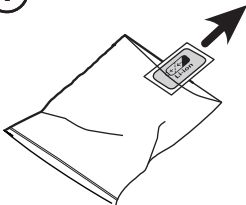
2

3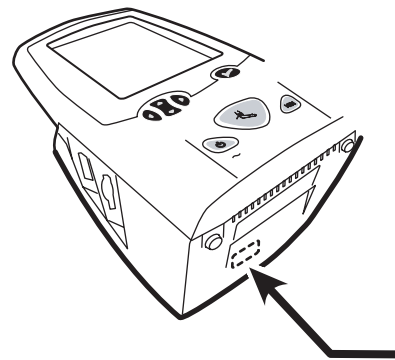

7

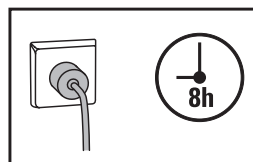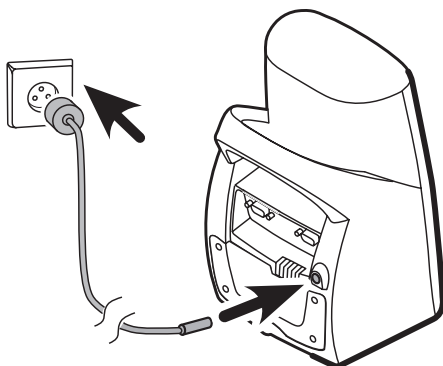
English

Set up the device

After recharging a dead battery or after disconnecting the battery for a few minutes, you must program the date and time screen.

To Enter the Internal Configuration Mode:

1. Turn the Spot LXi off (🔌).
2. Press and hold the **Select** (✔) and **Power** (🔌) buttons for 5 seconds. The device enters the Internal Configuration Mode and the Configuration Menu screen appears on the display.

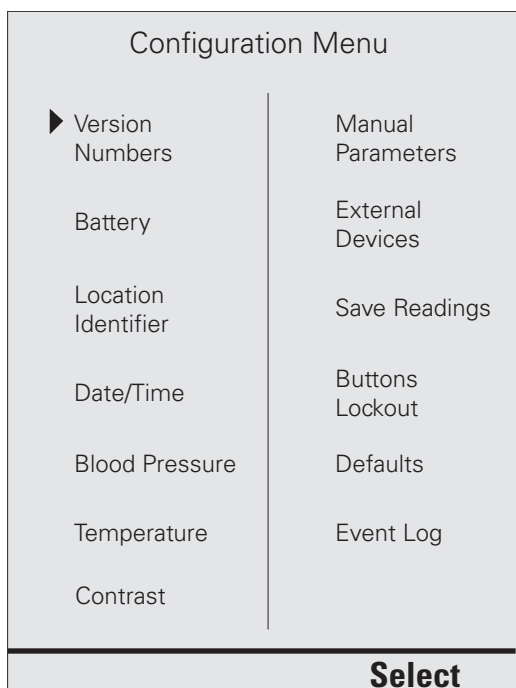3. Use the **Navigation** (⏪ ⏩) buttons to move through the menu options and press the **Select** (✔) button to access the options or accept a change.
4. Press the **Power** (🔌) button to exit the Internal Configuration Mode.

Date/Time Menu Options

Setting	Description
Date Format	Displays the date in one of the following styles: <ul style="list-style-type: none">• mm/dd/yyyy example: July 16, 2005 = 07/16/2005• dd/mm/yyyy example: 16 July 2005 = 16/07/2005
Date	Changes the date on the Display Window and in patient readings. If a location is entered the date will not appear on the Display Window; the location will.
Time Format	Displays the time in one of the following styles: <ul style="list-style-type: none">• 12-hour example: 5:00 PM• 24-hour example: 17:00
Time	Changes the time on the Display Window.

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