 Welch Allyn Limited
Navan Business Park
Dublin Road
Navan, County Meath
Republic of Ireland
www.welchallyn.com

Mat. 725388, Ver. A
DIR 80021380 Ver. A
Revision date: 2015-06

Dear Welch Allyn CP 50™ or CP 50 Plus™ 12-lead electrocardiograph owner:


Enclosed is a new software update, version 1.31.04, to improve the functionality of your electrocardiograph.

Improvements include:

- Automatic mains filter detection

Overview

This document describes the steps required to correctly update CP 50 devices to the latest software version using the UpdateUtility. It is important that you follow the following flow chart to identify the correct section to use when performing your update. The flow chart will determine the correct software update kit to use.

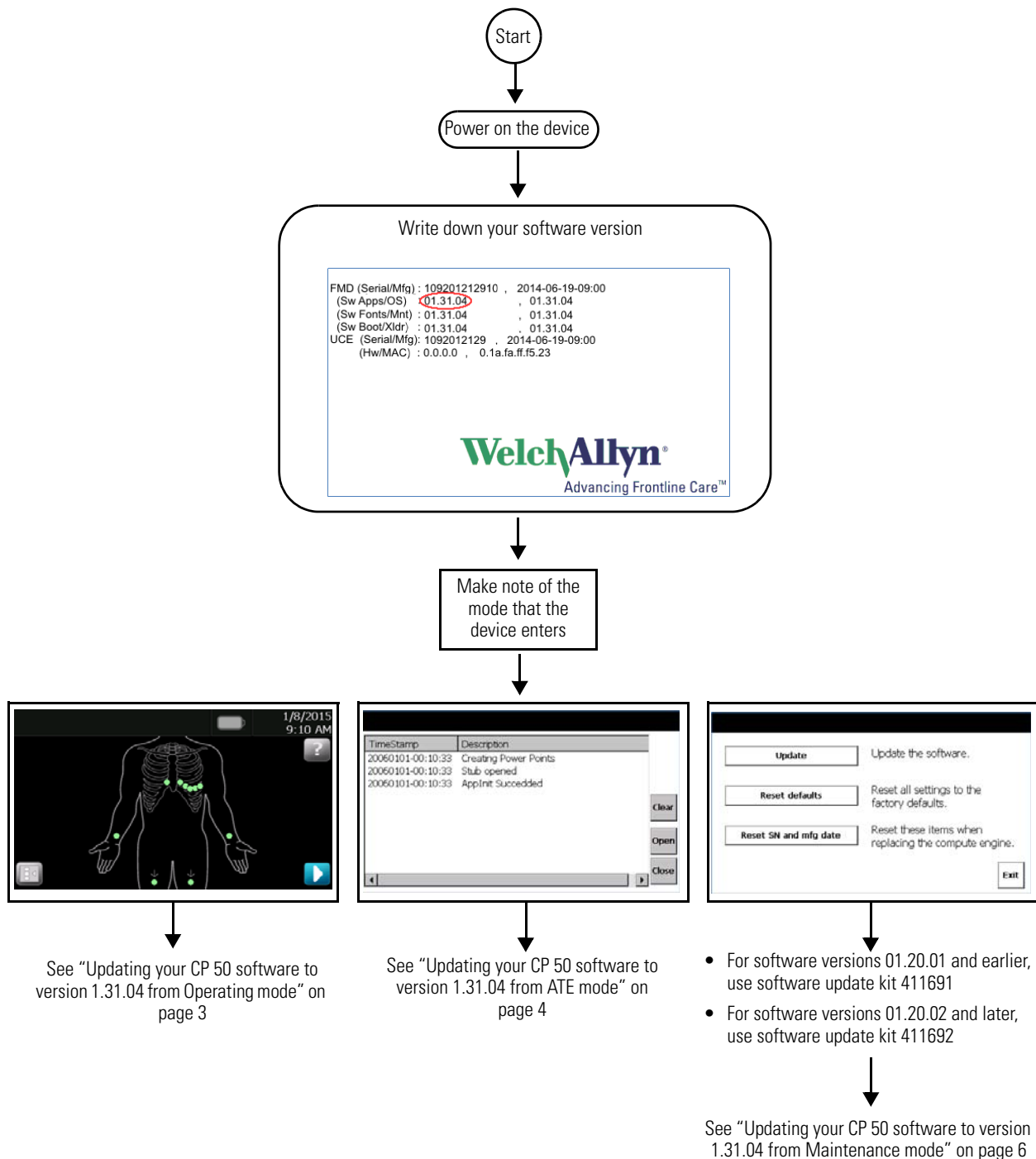Updating your CP 50 software to version 1.31.04 from Operating mode

Caution Data loss risk. Do not power down or interrupt the update process.

Before you begin

To perform the update, you will need the following:

- The USB drive (part number 411691) that contains the update utility.
- The CP 50 you plan to update.
- A cable to connect the CP 50 to AC power.
- An AC power outlet.

Update the software

1. Make sure the system is powered down. To power down the CP 50, press and hold the power button for 6 seconds.
2. Connect the CP 50 to AC power.
3. Plug the USB drive (part number 411691) into the CP 50.
4. Power up the CP 50 by pressing the power button. The software update begins after the device powers up.

Note The device will restart multiple times if the current software version 01.20.03 or later. The software update will operate on all earlier versions.

5. When the software update completes, the device might prompt you for the serial number and the manufacturing date. Enter these values and click **Update**.

Note The serial number is located on the label in the rear of the device. The current date can be entered if the manufacturing date is unavailable.

6. Remove the USB drive after the update is complete. Press **OK** to restart.

Caution Do NOT remove AC power from the device during the restart.

7. Once the device has completed the restart process, select your language.

Verify the software version:

- a. Go to **Menu > Settings (System tab) > About**.
- b. Verify that the correct software version is listed for the CP 50.


Updating your CP 50 software to version 1.31.04 from ATE mode

Caution Data loss risk. Do not power down or interrupt the update process.

Before you begin

In ATE mode, the following screen is displayed when the device starts up.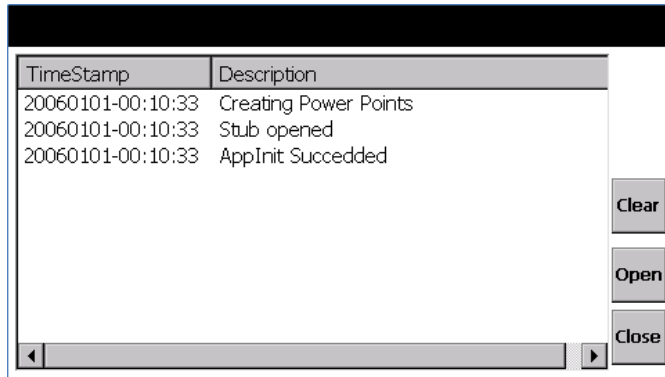

To perform the update, you will need the following:

- The USB drive (part number 411691) that contains the update utility.
- The CP 50 you plan to update.
- A cable to connect the CP 50 to AC power.
- An AC power outlet.

Update the software

1. Make sure the system is powered down. To power down the CP 50, press and hold the power button for 6 seconds.
2. Connect the CP 50 to AC power.
3. Plug the USB drive (part number 411691) into the CP 50.
4. Power up the CP 50 by pressing the power button. The software update begins after the device powers up.

Note The device will restart multiple times if the current software version 01.20.03 or later. The software update will operate on all earlier versions.

5. When the software update completes, the device might prompt you for the serial number and the manufacturing date. Enter these values and click **Update**.

Note The serial number is located on the label in the rear of the device. The current date can be entered if the manufacturing date is unavailable.

6. Remove the USB drive after the update is complete. Press **OK** to restart.

Caution Do NOT remove AC power from the device during the restart.

7. Once the device has completed the restart process, select your language.

Verify the software version:

- a. Go to **Menu > Settings (System tab) > About**.
- b. Verify that the correct software version is listed for the CP 50.


Updating your CP 50 software to version 1.31.04 from Maintenance mode


Caution Data loss risk. Do not power down or interrupt the update process.

Before you begin

In Maintenance mode, the following screen is displayed when the device starts up.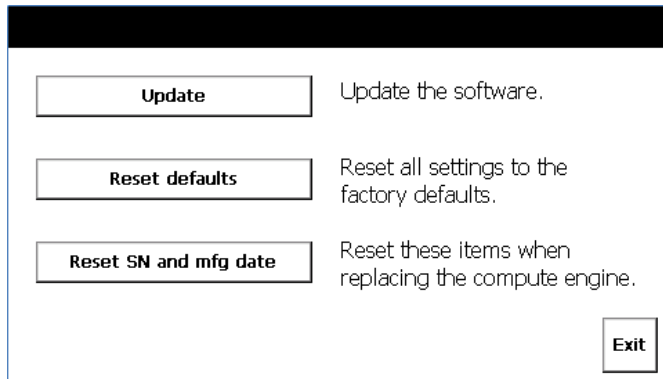

Note Ensure that you make note of the device software version displayed on the splash screen.

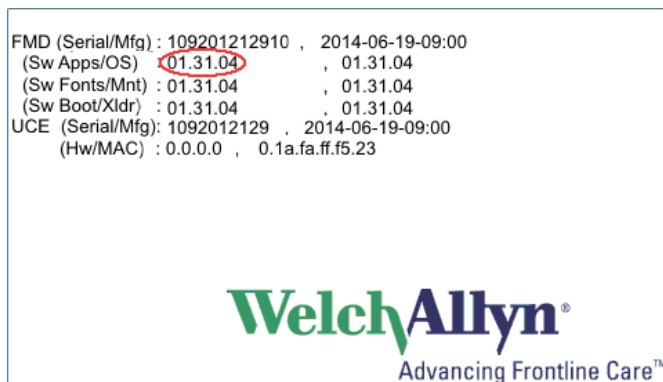Note If the current version of the device is 01.20.01 or earlier, use the USB drive with the part number of 411691. If the current version of the device is 01.20.02 or 01.20.03, use the USB drive with the part number 411692.

To perform the update, you will need the following:

- The USB drive that contains the update utility.
- The CP 50 you plan to update.
- A cable to connect the CP 50 to AC power.
- An AC power outlet.

Update the software

1. Make sure the system is powered down. To power down the CP 50, press and hold the power button for 6 seconds.
2. Connect the CP 50 to AC power.
3. Plug the USB drive into the CP 50.
4. Power up the CP 50 by pressing the power button.
5. Press **Update**.
6. Select the Xloader and press **Next**.
7. The screen displays the current and new versions. Press **Update**.
8. After update completes, press **OK** followed by the **Back** button. Repeat step 5 and step 6 for the remaining binary files:
 - Bootloader
 - Maintenance Image
 - Main OS
 - Main Fonts
 - Main Applications
9. After loading all the binary images, press **Next**.
10. The system is at the Maintenance screen. Press **Exit** and the system will restart within two seconds.

Note The device will restart multiple times if the current software version 01.20.03 or later. The software update will operate on all earlier versions.

11. When the software update completes, the device might prompt you for the serial number and the manufacturing date. Enter these values and click **Update**.

Note The serial number is located on the label in the rear of the device. The current date can be entered if the manufacturing date is unavailable.

12. Remove the USB drive after the update is complete. Press **OK** to restart.

Caution Do NOT remove AC power from the device during the restart.

13. Once the device has completed the restart process, select your language.

Verify the software version:

- a. Go to **Menu > Settings (System tab) > About**.
- b. Verify that the correct software version is listed for the CP 50.

Welch Allyn Limited
Navan Business Park
Dublin Road
Navan, County Meath
República da Irlanda
www.welchallyn.com

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