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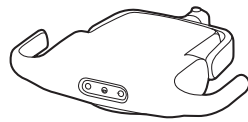
手册部件号 726221
DIR 80022018 Ver A
© 2017 Welch Allyn

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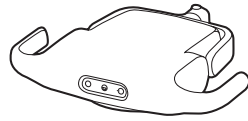
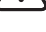
© 2014 Welch Allyn

Symbol Definitions

-  See the accompanying manual
-  Caution: High Intensity Light
-  Use indoors
-  Power on/off
-  Warning
-  Caution
-  Full power
-  Half power
-  Temperature limits
-  Atmospheric pressure limitation
-  Humidity limitation

Warnings and cautions

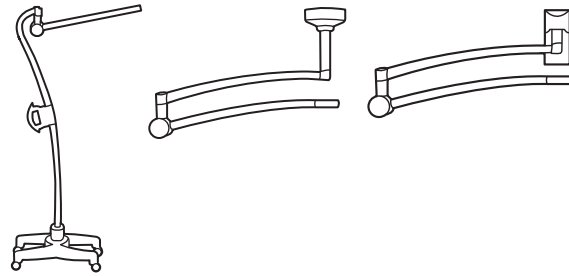
-  **WARNING:** Read and understand the instructions prior to use of this product. Failure to understand the operating requirements for this product may result in injury to yourself, the patient and/or may damage the instrument.
-  **WARNING:** Do not operate this product in the presence of flammable anesthetics or other flammable substances in combination with air, oxygen-enriched environments, or nitrous oxide; explosion can result.
-  **WARNING:** Illumination is intense. Do not stare at the light source in operation. May be harmful to eyes. To minimize risk of harm to the eyes from optical radiation hazards, avoid looking at bright light sources and their reflections, and protect eyes where normal pupil sizes and aversion responses are not present.
-  **WARNING:** Use the examination light in its intended working range of 24 in (60 cm) to 40 in (100 cm). Exposures at closer distances may be harmful to skin. To minimize risk of harm to the skin from optical radiation hazards, minimize illumination intensity at the tissue examination site, minimize exposure times, and take additional precautions when skin sensitivity has been altered through tissue trauma or the use of anesthesia.
-  **WARNING:** This is an EMC Class A product. In a hospital/physician's office environment this product may cause radio interference in which case the user may be required to take adequate measures.
-  **WARNING:** Do not modify this equipment. Any modification of this equipment can lead to patient injury. Any modification of this equipment voids the product warranty.
-  **CAUTION:** Only authorized Welch Allyn Service Centers can perform service on this product.
-  **CAUTION:** There are no user serviceable/replaceable parts for the luminaire.
-  **CAUTION:** Rx only.

Intended use

The Welch Allyn GS 900 is designed to meet the various needs of the physician's office, hospital environment and specialist's office. It is not intended for rendering diagnosis or surgery.

Mounting

Refer to the instructions supplied with the mounting system to ensure proper assembly.

 **WARNING:** Assembly of the mounting system must be carried out by a technician or another comparably qualified person.

 **WARNING:** Do not place objects on the luminaire.

Cleaning

 **WARNING:** Electrical shock. Before installation, turn off the product, disconnect the power, and ensure that the product cannot be switched on again accidentally.

Luminaire

Clean with a damp cloth using warm water and mild detergent or one of the following solutions:

- 70 percent isopropyl alcohol
- 10 percent mild bleach solution

Wipe dry with a clean cloth.

Troubleshooting

	Possible cause	Solution
	Power supply fault. No power supplied to light.	Verify all cables are connected correctly.
	Light is not functioning correctly	Call Welch Allyn Customer Service.

Warranty

Five years.

Specifications

Electrical

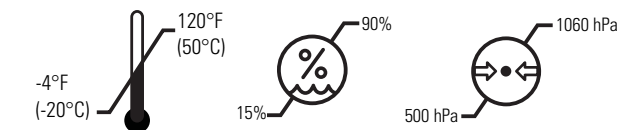
Input: 100-240 V~ 50Hz-60Hz, 1.25A max

Environment

Operating:

Transport/storage:

Equipment classification

Safety Class I, Continuous Operation

Additional technical information

www.welchallyn.com/gsseries

Approvals

IEC 60601-1, UL 60601-1, IEC 60601-1-2, CAN/CSA C22.2 No 601.1-M90, AS/NZS3200

 The CE mark on this product indicates that it has been tested and conforms with the provisions noted in the 93/42/EEC Medical Device Directive.

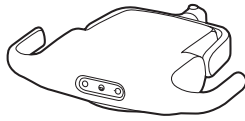
 Welch Allyn Inc.
4341 State Street Road
Skaneateles Falls, NY 13153-0220 USA
Tel.: + 1 315 685 4560
Fax: + 1 315 685 3361

EC REP Welch Allyn Ltd.
Dublin Road, Navan
County Meath, Republic of Ireland.
Tel.: +353 46 9067700
Fax: +353 46 9067755

NOTE: This product complies with current required standards for electromagnetic interference and should not present problems to other equipment or be affected by other devices. As a precaution, avoid using this device in close proximity to other equipment.

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