

Mobile Stand Assembly Instructions

Welch Allyn Technical Support

www.welchallyn.com/support


Mat. No. 727874, 80023177 Ver. A

Revision date: 2018-03

© 2018 Welch Allyn, Inc.



(91)727874

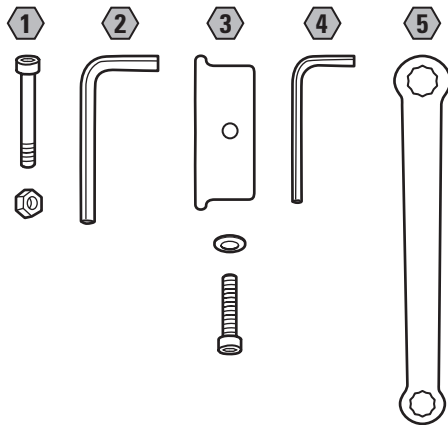


A

#

4800-60, 4900-60

Accessory Cable Management
Mobile Stand

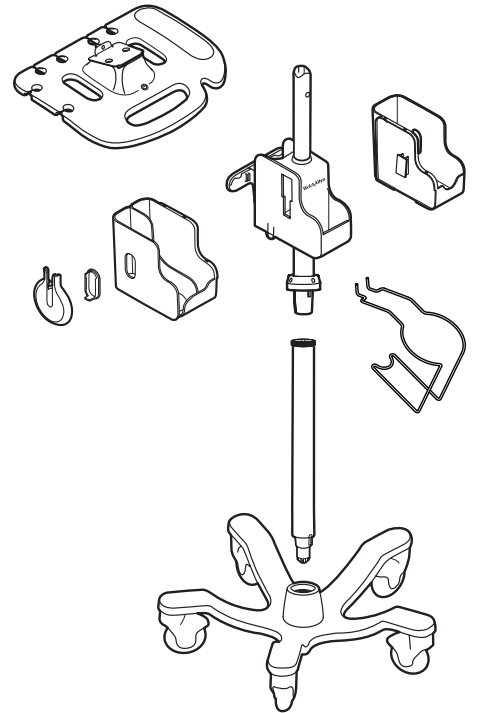


B

#

7000-APM

Accessory Power Management
Mobile Stand

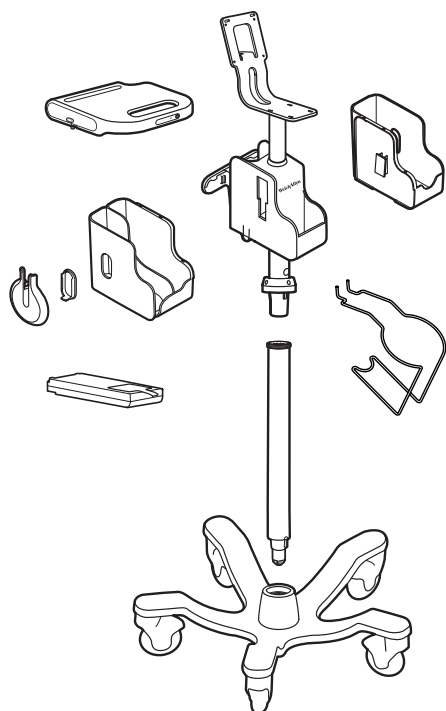


C






#

7000-MWS

Mobile Work Surface Stand

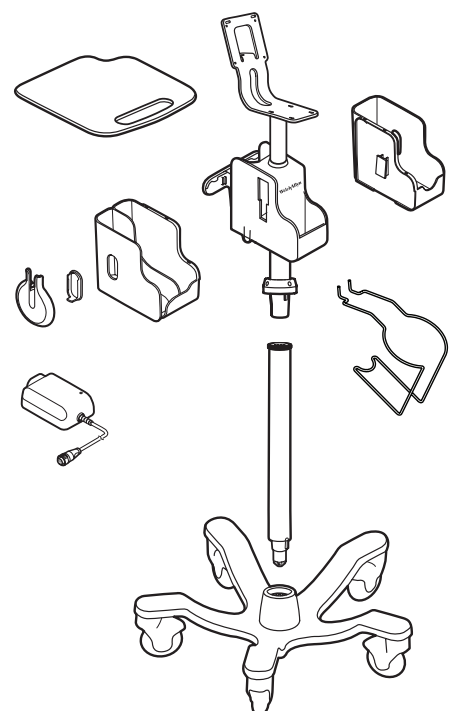


1 A, B, C

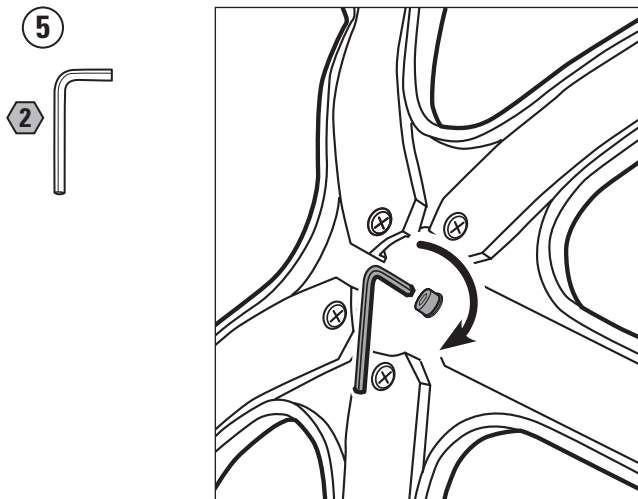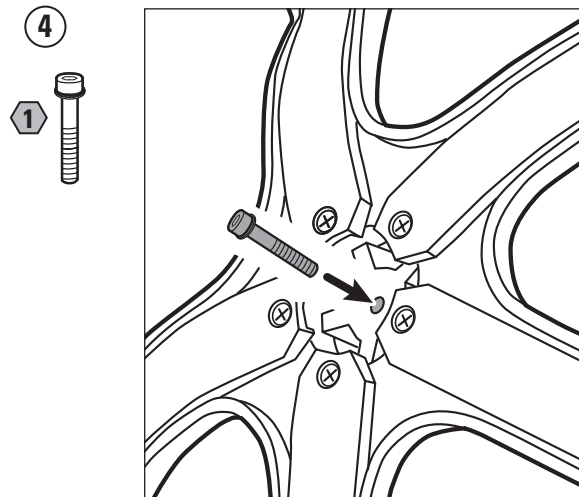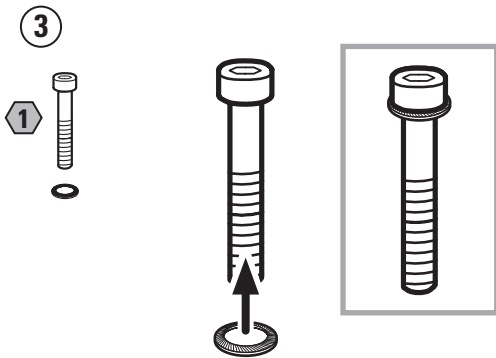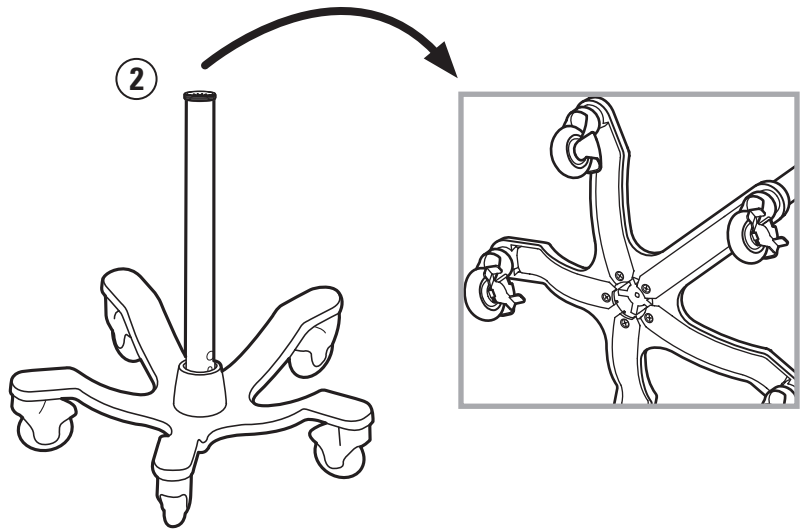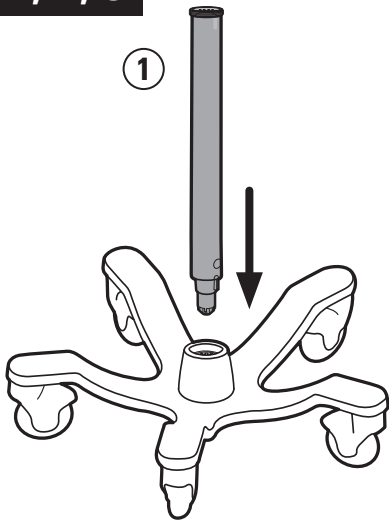

80+/-10 in-lb. (9.04+/-1.13 Nm)

- Tighten the screw with the wrench provided.
- Serrer la vis au moyen de la clé fournie.
- Schraube mit dem mitgelieferten Schraubenschlüssel festziehen.
- Stringere la vite con la chiave fornita.
- Apriete el tornillo con la llave que se facilita.
- Strângeți șurubul cu cheia furnizată.
- Húzza meg a csavart a mellékelt kulcs segítségével.
- Σφίξτε τη βίδα με το γαλλικό κλειδί που παρέχεται.
- Pomocí dodaného klíče dotáhněte šroub.
- Zategnite zavrtanj priloženim ključem.
- Priloženim ključem pritegnite vijak.
- S priloženim ključem do konca privijte vijak.







2 A, B, C

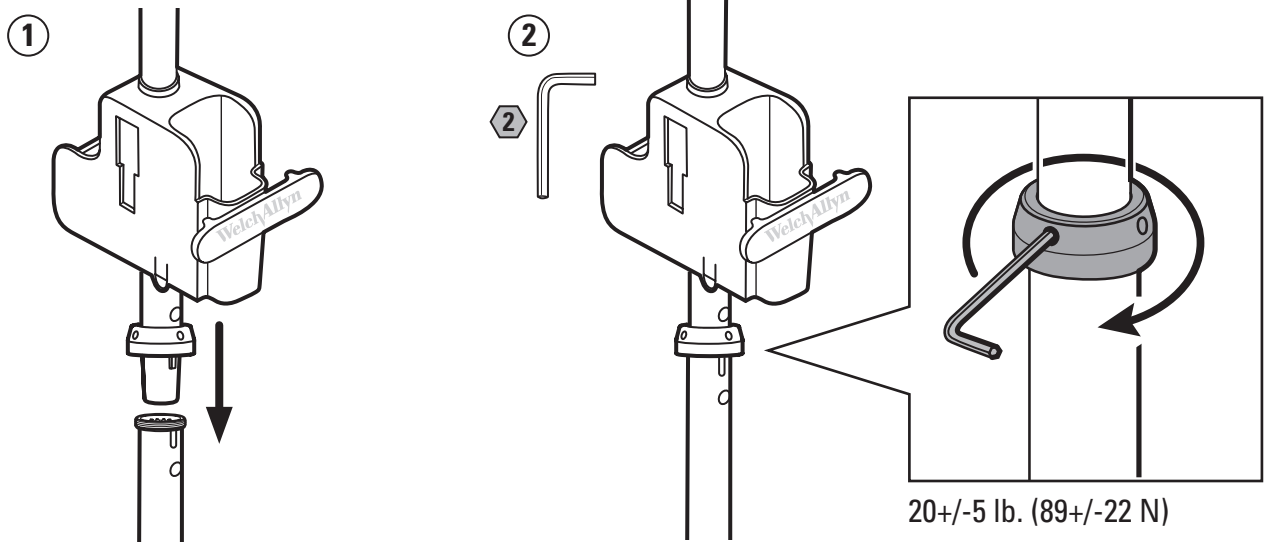


- Tighten the collar clockwise around the pole with the wrench provided.
- Serrer le collier dans le sens des aiguilles d'une montre autour de la potence au moyen de la clé fournie.
- Manschette mit dem mitgelieferten Schraubenschlüssel im Uhrzeigersinn auf dem Ständer festschrauben.
- Stringere il collare intorno all'asta in senso orario con la chiave fornita.
- Apriete el collar en el sentido de las agujas del reloj alrededor del portasueros con la llave que se facilita.
- Strângeți colierul spre dreapta în jurul stâlpului cu cheia furnizată.
- Húzza meg a gyűrűt a rúdön a mellékelt kulcs segítségével.
- Σφίξτε το κολλάρο δεξιόστροφα γύρω από τον στύλο με το γαλλικό κλειδί που παρέχεται.
- Pomocí dodaného klíče utáhněte límeč kolem tyče po směru hodinových ručiček.
- Pomoću priloženog ključa zategnite prsten oko stalka u smeru kazaljke na satu.
- Priloženim ključem pritegnite produžetak oko pola u smjeru kazaljke na satu.
- S priloženim ključem privijte objemko v smeri urinega kazalca.



3 A, B, C

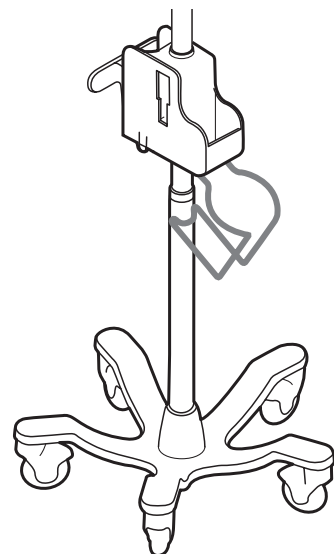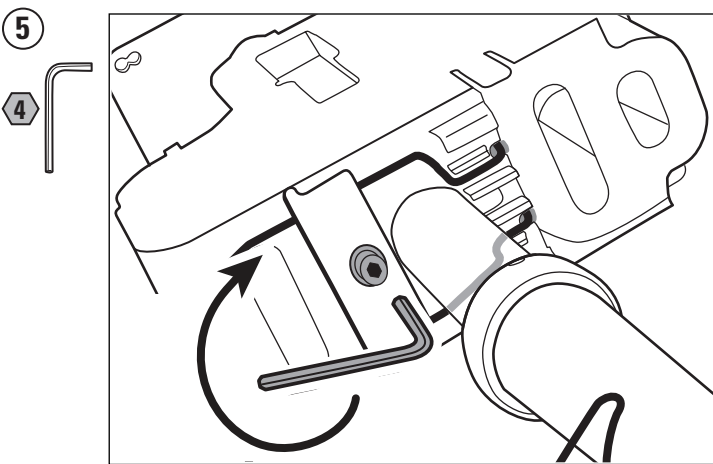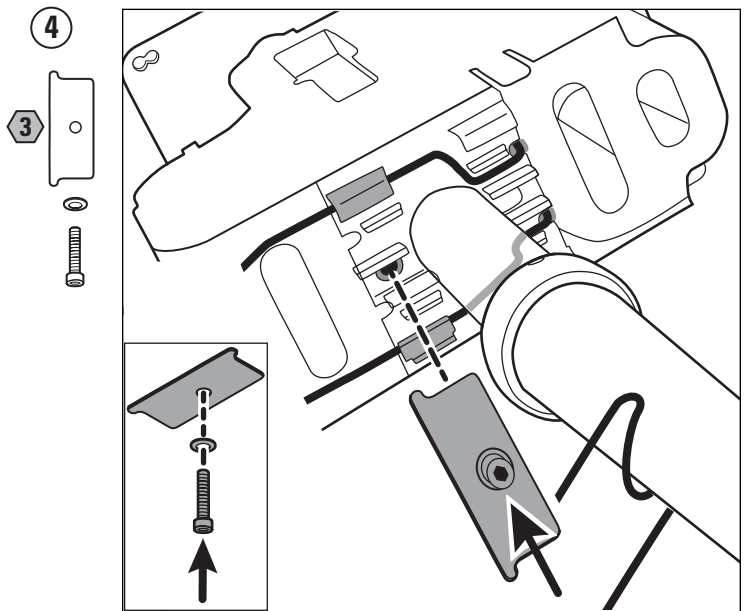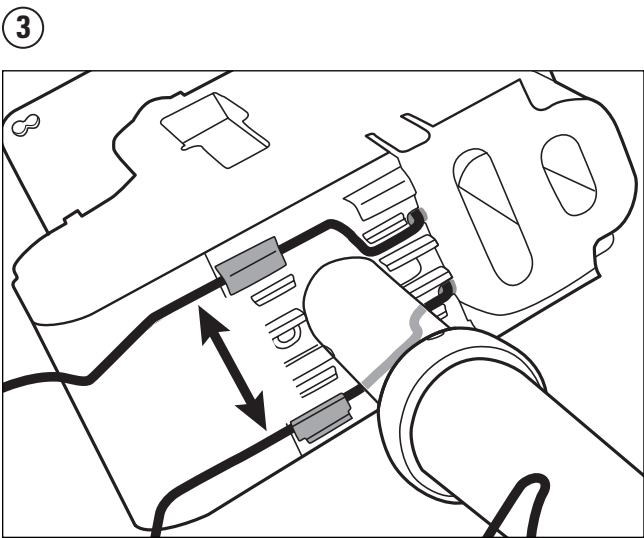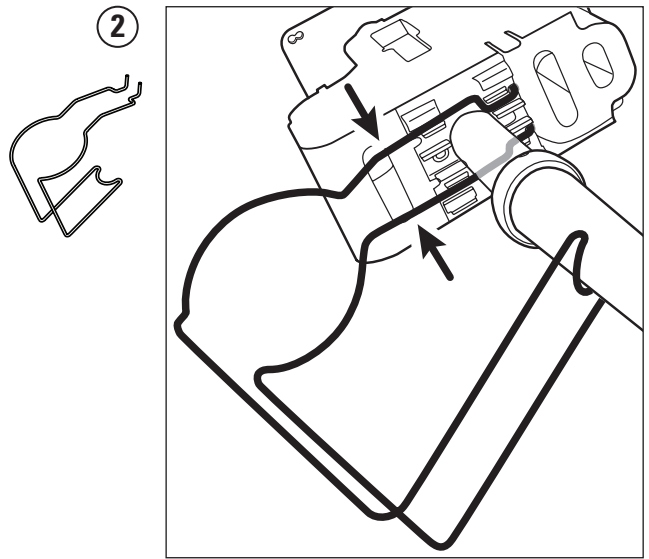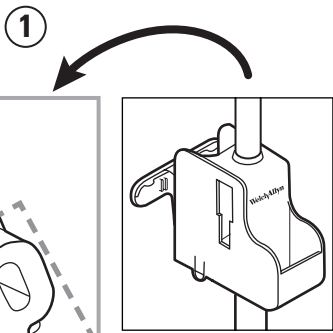

4 A, B, C

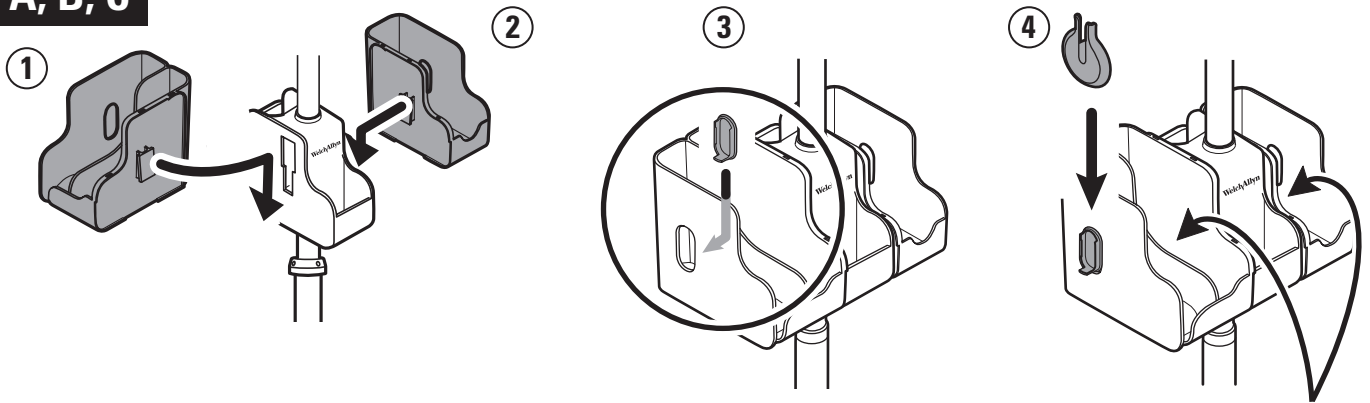


- Maximum safe working load limits.
- Charge de travail de sécurité maximale.
- Maximale sichere Tragfähigkeit.
- Limiti del carico di lavoro massimo in sicurezza.
- Carga de trabajo segura máxima.
- Limite pentru sarcina maximă sigură de lucru.
- Maximális biztonságos üzemi terhelés határértékei.
- Όρια μέγιστου ασφαλούς φορτίου εργασίας.
- Maximální limity bezpečné pracovní zátěže.
- Granice maksimalnog bezbednog radnog opterećenja.
- Ograničenja maksimalnog sigurnog radnog opterećenja.
- Najvišje varne delovne obremenitve.




5 A

4800-60, 4900-60 Accessory Cable Management Mobile Stand

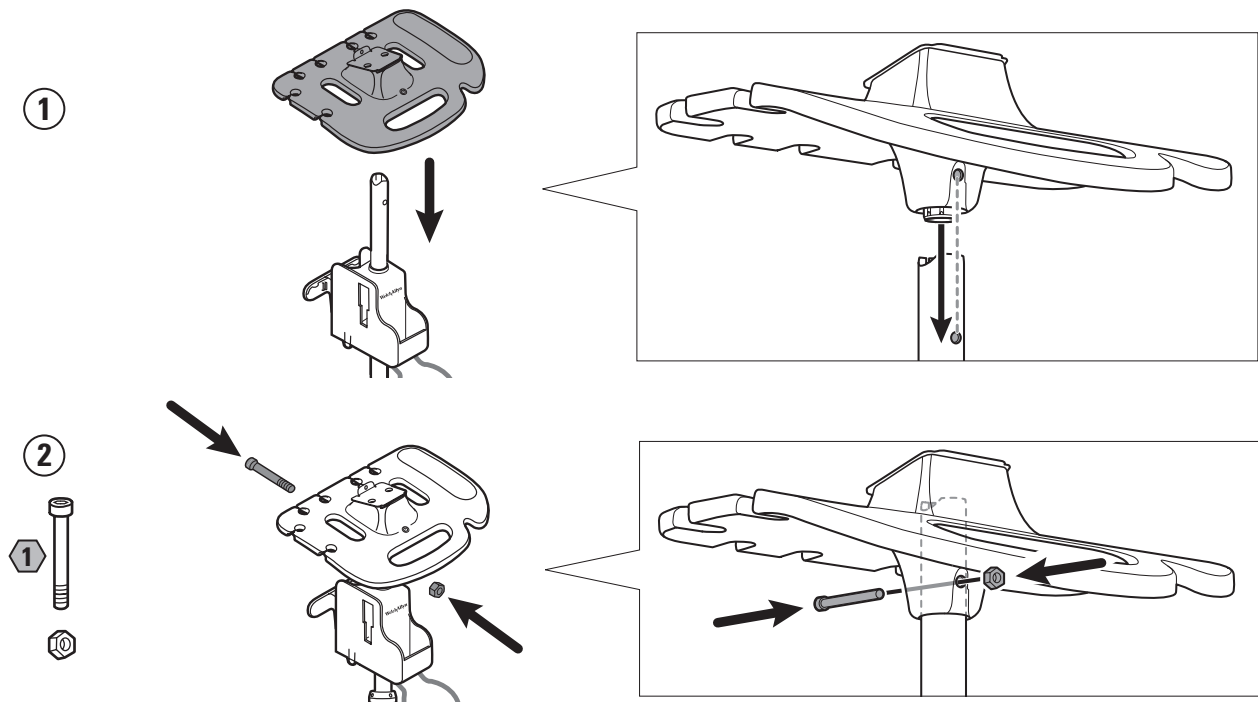


- Tighten the screw with the wrench provided.
- Serrer la vis au moyen de la clé fournie.
- Schraube mit dem mitgelieferten Schraubenschlüssel festziehen.
- Stringere la vite con la chiave fornita.
- Apriete el tornillo con la llave que se facilita.
- Strângeți șurubul cu cheia furnizată.
- Húzza meg a csavart a mellékelt kulcs segítségével.
- Σφίξτε τη βίδα με το γαλλικό κλειδί που παρέχεται.
- Pomocí dodaného klíče dotáhněte šroub.
- Zategnite zavrtanj priloženim ključem.
- Priloženim ključem pritegnite vijak.
- S priloženim ključem do konca privijte vijak.

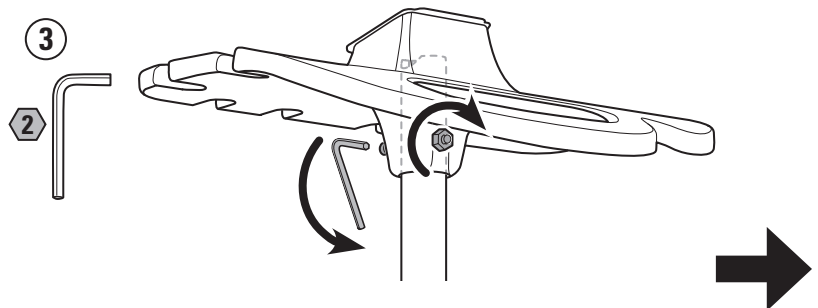


5 B

7000-APM Accessory Power Management Mobile Stand

1

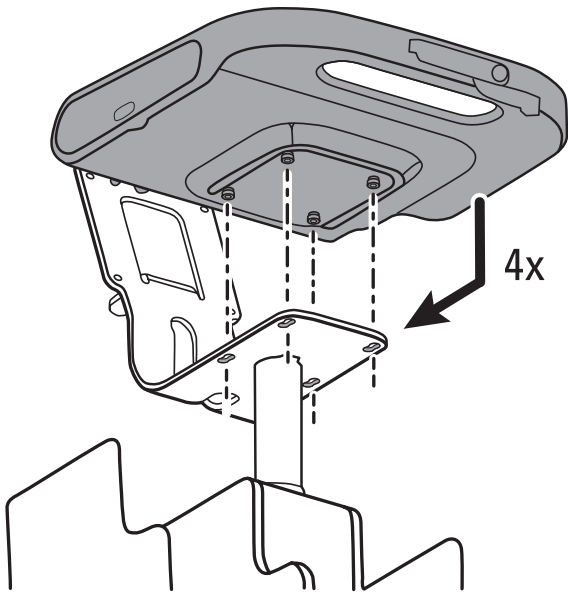

2

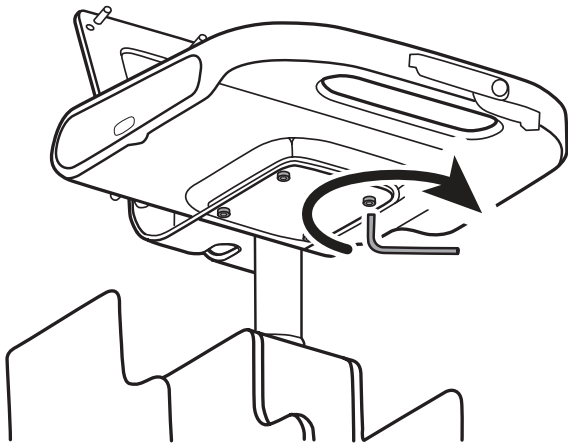


5 C


7000-MWS Mobile Work Surface Stand

1


5

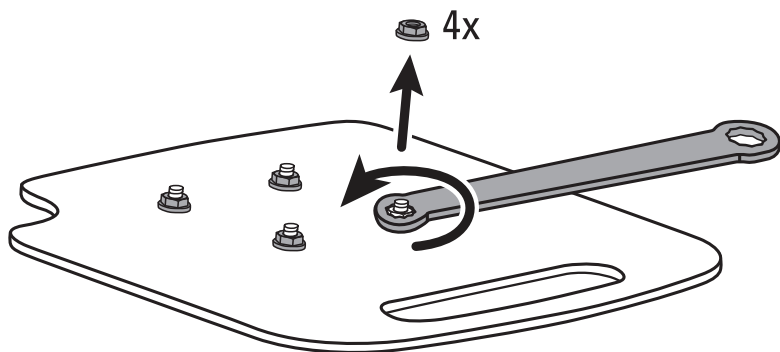

2

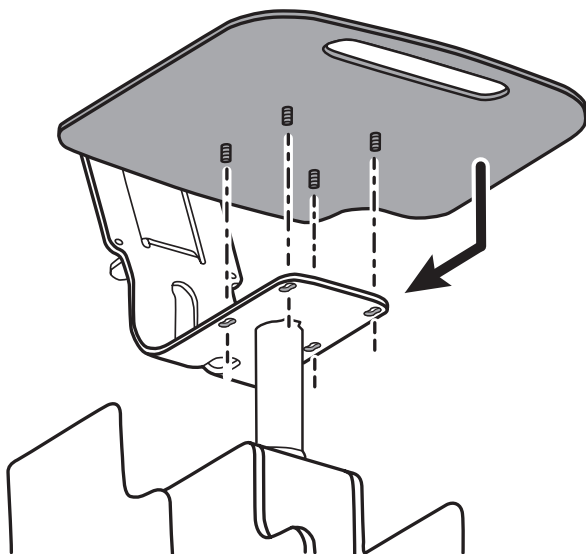







3

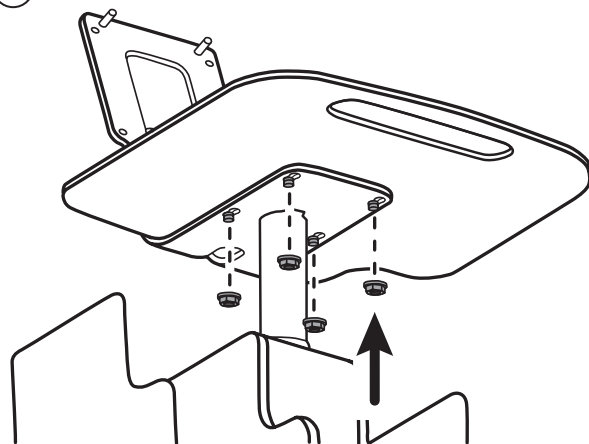

4

5

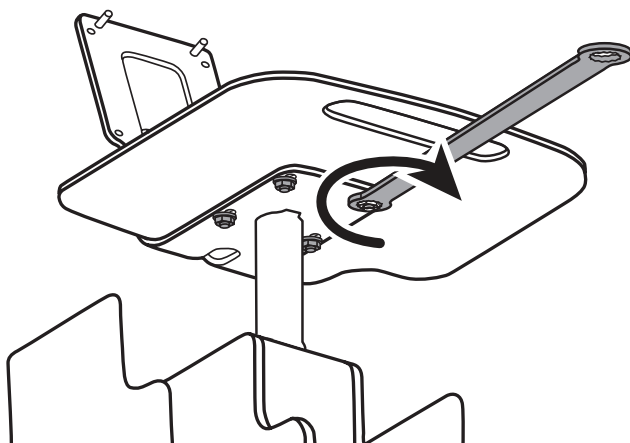

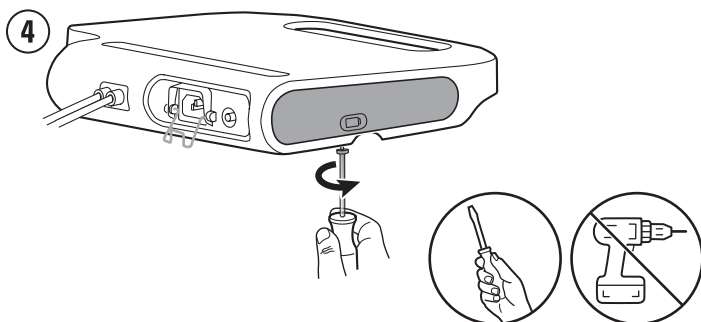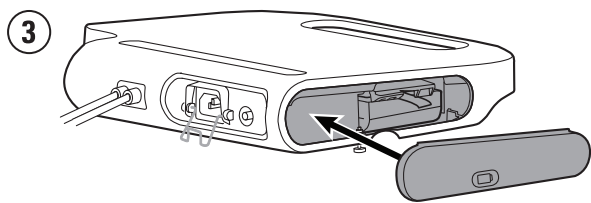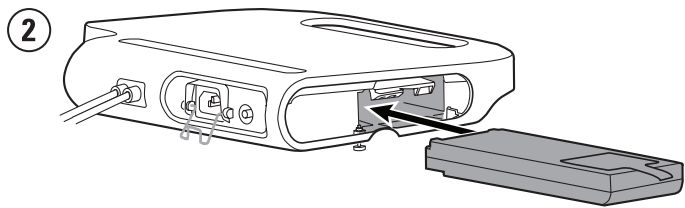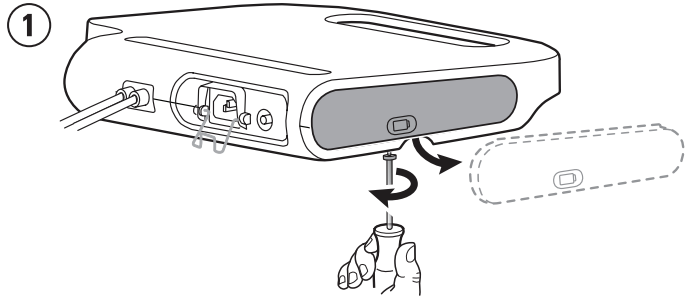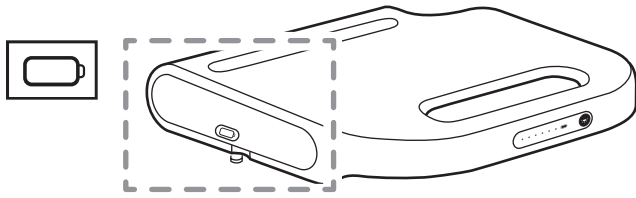6 B# **7000-APM** Accessory Power Management Mobile Stand**6 C**# **7000-MWS** Mobile Work Surface Stand