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**The Ownership of Comfort and Precision**

Welch Allyn presents the Professional™ — a full-range stethoscope that offers extraordinary comfort and superior acoustics. Comfort sealing eartips and light weight make the Professional an exceptionally comfortable stethoscope.

The Professional stethoscope provides medical professionals with a broad range of auscultation capabilities, from the very low pitch (20-100 Hz) of certain heart murmurs to the higher frequencies (100-500 Hz) of aortic and pulmonic insufficiencies. The Professional stethoscope augments and clarifies the sound for accurate diagnosis.

The Professional stethoscope enhances your professional appearance as well as your diagnostic capabilities. Its light weight and flexible tubing make it perfect for slipping into a side pocket, or carrying around the neck.

**Chestpieces**

The Professional stethoscope offers both a diaphragm and bell which you select by holding the chestpiece stem in one hand and rotating the chestpiece with the other hand until a click is heard or felt.


**Diaphragm**

"The workhorse." Excellent for all sounds and murmurs. Best for higher frequency heart and lung sounds. Also ideal for identifying split sounds, systolic clicks and ejection sounds.

**Bell**



Provides excellent detection and clarity for low and medium frequency heart sounds and murmurs (such as a faint gallop or diastolic rumble).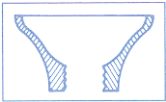

**Headset Adjustment**

Your new Professional stethoscope is designed to give you the maximum ability to adjust to your personal comfort and acoustic fit.

1. Select the eartip that you prefer. For maximum acoustic seal and comfortable fit, the Professional stethoscope is equipped with comfort sealing eartips. The comfort sealing tip is universal in size, and designed to conform to the ear canal opening to block out ambient noise. A set of medium-sized plastic eartips is also included. The eartips are threaded and screw onto the binaural tubes for a secure connection.
2. To adjust the angle of the binaurals, hold the tubing in one hand and rotate each binaural to the correct angle.
3. Make sure that the binaural tubes are pointed slightly forward into the ear canal.
4. To reduce spring tension, grasp each eartube at its bend and gently pull outwards.
5. To increase spring tension, squeeze the eartubes together.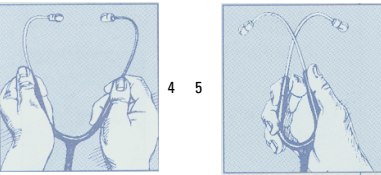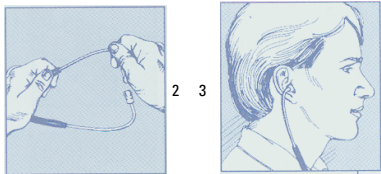

**Care and Maintenance**



It is important to clean and disinfect your stethoscope to prevent possible cross-contamination.

Binaurals and tubing can be wiped down with alcohol or mild soapy water.

Eartips may be removed for thorough cleaning with alcohol or mild soapy water.

Diaphragm replacement: Under normal use you should rarely need to remove the rim and diaphragm for cleaning purposes. The exterior of the diaphragm can be cleaned with alcohol or mild soapy water. However, if you do need to remove the diaphragm, follow the instructions below.

1. Holding the diaphragm side facing you, grip the retaining rim with thumb and index finger and push the rim off the edge of the metal chestpiece.
2. To reassemble, align the diaphragm on the chestpiece and push the rim upward.
3. Continue to totally fit the rim onto the chestpiece by aligning the parts in an upward motion until the rim completely snaps on.

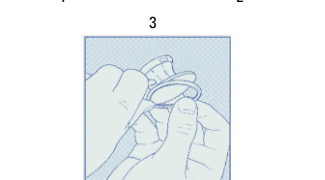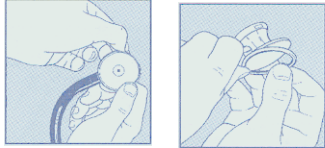The bell non-chill rim can be removed and replaced in a similar manner.

**Warranty**

Your Professional stethoscope comes with a five-year warranty against parts and manufacturing defects. Should a material or workmanship defect be discovered, Welch Allyn will repair or replace the product at no charge. This warranty does not cover damage in shipment or failure caused by tampering, obvious carelessness or abuse.

Welch Allyn®

Professional™  
Stethoscope

Directions for Use

WelchAllyn®

Advancing Frontline Care™

Mat'l No. 95P478 Rev. F

Welch Allyn USA (Headquarters)  
Tel: 1-315-685-4560 or 1-800-535-6663  
Fax: 1-315-685-0054

Welch Allyn Ireland (European Customer Service)  
Tel: (353) 46 90 67700  
Fax: (353) 46 90 67756

Welch Allyn Canada  
Tel: 1-905-890-0004  
Fax: 1-905-890-0008

Welch Allyn U.K.  
Tel: 0-207-365-6780  
Fax: 0-207-365-9694

Welch Allyn Italy  
Tel: (+39) 02-6968-2425  
Fax: (+39) 02-6963-3697

Welch Allyn Australia  
Tel: (+61) 2-9638-3000  
Fax: (+61) 2-9638-3500

Welch Allyn China  
Tel: (+86) 21 6327 9631  
Fax: (+86) 21 6327 9632

Welch Allyn Japan  
Tel: (+81) 3-219-0071  
Fax: (+81) 3-219-0074

Welch Allyn Singapore  
Tel: (+65) 291-0882  
Fax: (+65) 291-5780

Welch Allyn Germany  
Tel: (+49) 7477 9271 70  
Fax: (+49) 7477 9271 90

Welch Allyn France  
Tel: (+33) 60 09 33 66 or (+33) 55 69 58 49  
Fax: (+33) 60 09 67 97

Welch Allyn Spain  
Tel: (+34) 917499357  
Fax: (+34) 917499361

Welch Allyn South Africa  
Tel: (+27) 11 777 7509  
Fax: (+27) 11 777 7510

Welch Allyn, Inc.  
4341 State Street Road  
Skaneateles Falls, NY 13153  
Tel. 315.685.4560 Fax 315.685.3361 www.welchallyn.com

The CE mark on this product indicates that it is in conformity with the provisions of Council Directive 93/42/EEC.

European Regulatory Manager, Medical Division, Welch Allyn Ltd. Navan Business Park, Dublin Road, Navan, County Meath, Republic of Ireland  
Tel: +353 46 90 67700 Fax: +353 46 90 67756

© 2006 Welch Allyn, Inc. All rights reserved. No one is permitted to reproduce or duplicate, in any form, this manual or any part thereof without permission from Welch Allyn. Welch Allyn assumes no responsibility for any injury to anyone, or for any illegal or improper use of the product that may result from failure to use the product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual. Welch Allyn® is a trademark of Welch Allyn, Inc.

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