

Connecting DETECTO® Scales to Welch Allyn Devices

Bariatric



6857DHR
Waist-High
Bariatric Scales

Wheelchair



FHD144II
Stationary
Wheelchair Scales

6550
Fold-up Portable
Wheelchair Scales

BRW1000
Bariatric
Wheelchair Scales

Digital Column



8430
Waist-High
Medical Scales

6437DHR
Eye-Level
Medical Scales


Connex® Spot Monitor



Connex® Vital Signs Monitor 6000 Series



Connex® Integrated Wall System

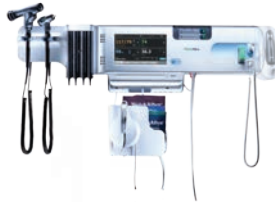


Easily Connect Your Welch Allyn Device to a DETECTO Scale



DETECTO SCALE



CONNECTIVITY KIT

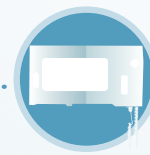

WELCH ALLYN
DEVICE W/ LICENSE
ACTIVATED



EMR

Welch Allyn Connectivity to DETECTO Professional Scales

The following table outlines the specific DETECTO scales that have been approved for use with both the Connex VSM 6000 Series (including the Connex IWS) and Connex Spot Monitor.

Model Number	Scale Description	Scale Family
6437DHR	Digital Physician Scale with Digital Height Rod	Digital Column
6449	Digital Scale with Height Rod and Handpost	Digital Column
6439	Digital Eye-Level Scale with Height Rod	Digital Column
6437	Digital Eye-Level Physician Scale	Digital Column
8437	Digital Waist-High Physician Scale	Digital Column
8430	Digital Waist-High Scale with Height Rod	Digital Column
8437S	Stainless Steel Scale, Waist-High Digital	Digital Column
8430S	Stainless Steel Scale, Waist-High Digital and Height Rod	Digital Column

6857DHR	Waist-High Stand-On Bariatric Scale — Digital Height Rod	Bariatric
6854-DHR	Waist-High Stand-On Bariatric Scale — Digital Height Rod	Bariatric

BRW1000	Portable Bariatric Wheelchair Scale	Wheelchair
6500	Portable Wheelchair Scale	Wheelchair
6550	Folding Portable Wheelchair Scale	Wheelchair
FHD-133-II	Geriatric/Bariatric Wheelchair Scale 36" X 36"	Wheelchair
FHD-144-II	Geriatric/Bariatric Wheelchair Scale 48" X 48"	Wheelchair
6495	Digital Stationary Wheelchair Scale	Wheelchair
CR-500D	"Roll-A-Weigh" Wheelchair Scale, 500 lb Capacity	Wheelchair
CR-1000D	"Roll-A-Weigh" Wheelchair Scale, 1,000 lb Capacity	Wheelchair

Scale Connectivity Kits for the following Welch Allyn devices:

Welch Allyn Connex Spot Monitor



Welch Allyn Connex Vital Signs Monitors



Welch Allyn Connex Integrated Wall System



License Required
<p>No No license required. Connectivity kit must be purchased from DETECTO</p>
<p>Yes License key included with purchase of connectivity kit from DETECTO</p>
<p>Yes License key included with purchase of connectivity kit from DETECTO</p>

Important Notes Regarding Acquisition and Setup of Scale Connectivity:

- Welch Allyn will not sell scales or connectivity kits/licenses.
- Connex Vital Signs Monitors require software version 1.7 or later to access the scale connectivity feature.
- Connex Spot Monitors require software version 1.2 or later to access the scale connectivity feature.
- With Connex Vital Signs Monitors and Connex Integrated Wall Systems, scale connectivity feature must be activated on Welch Allyn device using the Welch Allyn Service Tool and a license key shipped with the connectivity kit.
- Connex Spot Monitor does not require a connectivity license.
- Connectivity license must be purchased through DETECTO.

Important Notes Regarding Function of Welch Allyn Devices with DETECTO Scales:

- **Select DETECTO Scales:** Measure weight, allow for manual entry or digital capture of height and calculate BMI.
- **Select DETECTO Scales:** Have the ability to transfer values for weight, height and BMI from a DETECTO digital scale to a Welch Allyn Device.
- **Connex Spot Monitors with 1.2 software and Connex Vital Signs Monitors with 1.7 software or greater:** Accept and display weight, height and BMI when transmitted from the scale. (Connex monitors do not calculate BMI.) The monitors also allow for manual entry of weight, height and/or BMI.

Important Note Regarding EMR Connectivity:

EMR connectivity requires existing Welch Allyn device interface built by the specific EMR vendor for customer's specific EMR product and release. This interface must include each specific category (height, weight, BMI) for that information to transfer.

For more information on DETECTO scales,
please call 800.641.2008 or visit detecto.com/detecto-welch-allyn-connectivity-video.